



10 mL  
delivery volume

# Directions For Use

## Single Use Fluid Delivery Pump

See inside for instructions

Standard conditions are 25°C, 1.5 kPa. For flow rate variation due to other temperatures or back pressures visit [www.medecell.com/ratevariation](http://www.medecell.com/ratevariation) or call Med-e-Cell's technical support at 858-552-0781.

- NOT FOR HUMAN USE -

### Note:

- Do not use if the package has been damaged.
- Store at room temperature.

U.S. Patent #4,902,278. Others pending.

For Product Information and Technical Support phone : 858-552-0781  
web : [www.medecell.com](http://www.medecell.com)




10633 Roselle St., San Diego, CA 92121

2464r0

This package contains a single use Infu-Disk™ disposable pump with a Luer lock adapters.

### Precautions :

- **Do not apply excessive force on needle when filling.**
- **Do not obstruct the port on top of the start button.**
- **Do not re-use.**

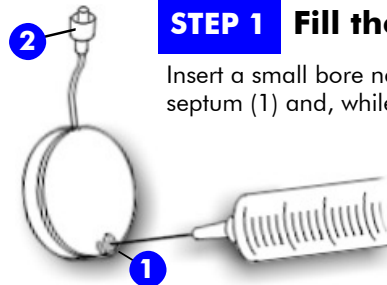

### STEP 1 Fill the Pump

Insert a small bore needle (20G to 22G) through the septum (1) and, while holding the pump as shown, Inject the fluid into the chamber until it appears at the tip of the Luer. **Make sure the fluid chamber is free of air bubbles.**

### STEP 2 Start the Pump

**Do not press the start button until ready to use.**

Start the pump by pressing the protruding start button (3) until flush with the button ring (4). Caution: A small amount of liquid (approximately 0.25 mL) will be expelled from the pump when the start button is pressed. The pump is now operating.

