



5 mL
delivery volume

Single Use Fluid Delivery Pump

See inside for instructions

Note:

- Do not use if package is damaged.
- Store at room temperature.

Standard conditions are 25°C, 1.5 kPa. For flow rate variation due to other temperatures or back pressures visit www.medecell.com/ratevariation or call Med-e-Cell's technical support at 858-552-0781.

- NOT FOR HUMAN USE -

Product Information and Technical Support
phone : 858-552-0781
web : www.medecell.com

U.S. Patent #4,902,278. Others pending.



10633 Roselle St., San Diego, CA 92121

2469r0


Directions For Use

This package contains a single use Infu-Disk™ disposable pump with a 22 gauge cannula.

Precautions :

- Do not apply excessive force on needle when filling.
- Do not obstruct the port on top of the start button.

STEP 1 Fill the Pump

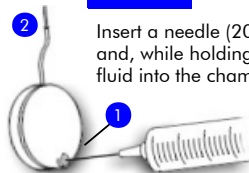

Insert a needle (20G to 22G) into the septum (1) and, while holding the pump as shown, inject the fluid into the chamber until it appears at the tip of the cannula (2). Make sure the fluid chamber is purged of air bubbles.

STEP 2 Start the Pump

Do not press the start button until ready to use.

Start the pump by pressing the protruding start button (3) until flush with the button ring (4). The pump is now operating.

